



INFLATION USING THE PAXISTM RAPID INSTILLATION KIT



Refer to the complete IFU for full Instructions for Use.

Connect the luer connector of the rapid instillation tubing to the red back-check valve on the blue stopcock.



Insert the spike into the bag of preferred sterile liquid as per institutional protocol.




2.

Connect a syringe to the remaining available port of the blue stopcock.

Note: The blue stopcock tap position remains unchanged during inflation.




Draw back the syringe plunger to fill the syringe with fluid.

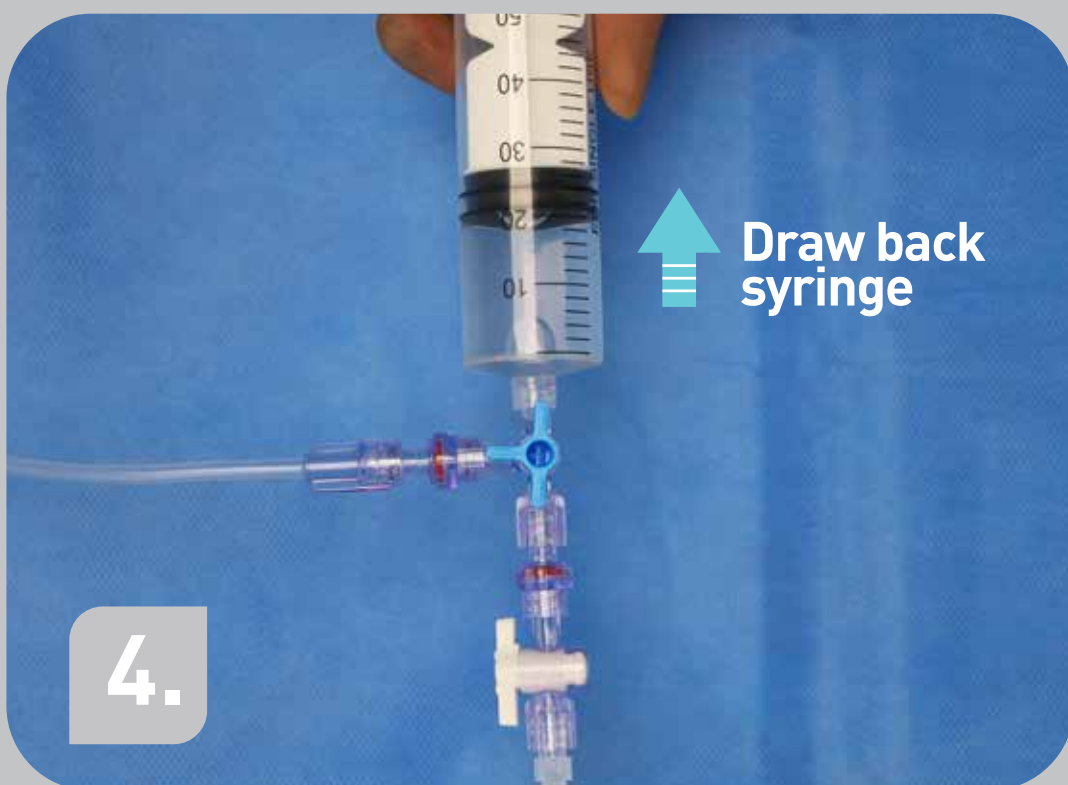

Depress the syringe plunger to deliver the fluid into the balloon.

Note: The white stopcock tap remains unchanged in the open position during the inflation procedure.

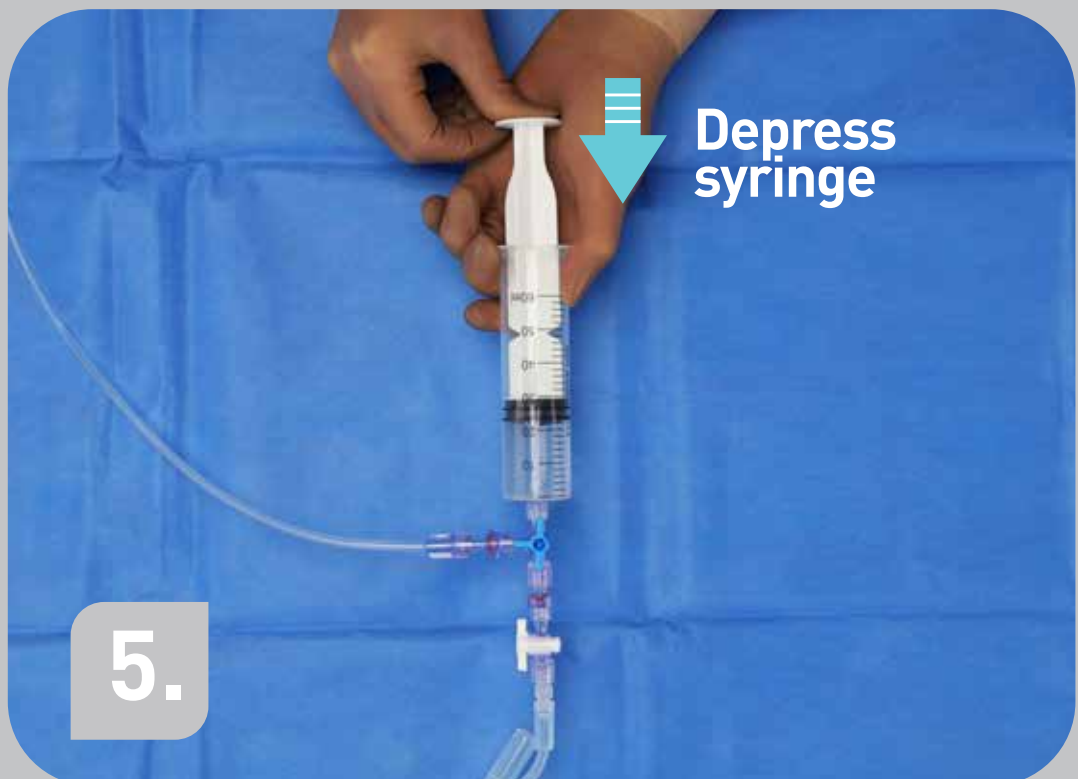

Repeat this process of drawing fluid into the syringe and pushing it into the balloon while monitoring the volume on the fluid bag until the predetermined amount is reached.

