


# Urine Drainage Bag


## **i** INSTRUCTIONS FOR USE




**1.** Perform hand hygiene.

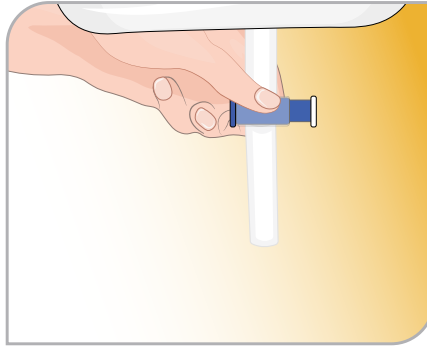


**2.** Remove bag from the packaging and check that the T-Tap at the distal end of the bag is closed.

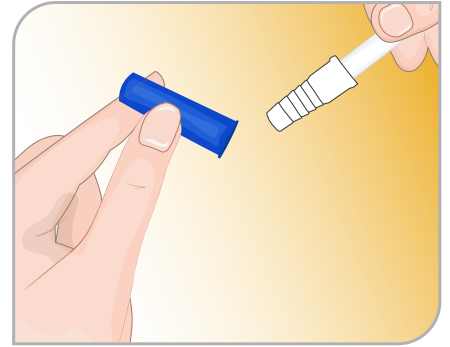

**3.** Using aseptic technique, remove cap from the drainage tubing and connect directly into the funnel of the urinary catheter.

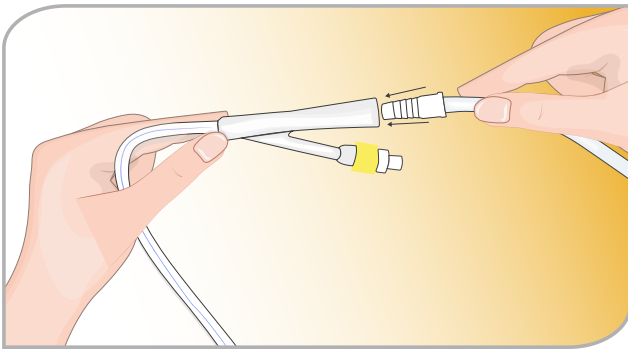

**4.** Ensure the connection is secure.

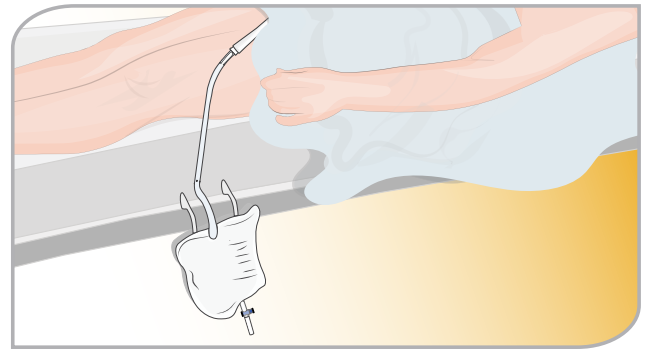

**5.** Hang the bag at or below the patient's bladder level.

## TO EMPTY THE URINE BAG



**1.** Perform hand hygiene.




**2.** Place a receptacle directly under the T-Tap of the Urine Bag.

# Urine Drainage Bag



- 3.** Holding the T-Tap in one hand, with fingers on either side of the tap, gently push against the mechanism to slide the tap across the T-connection to open the tap to allow the urine to empty.

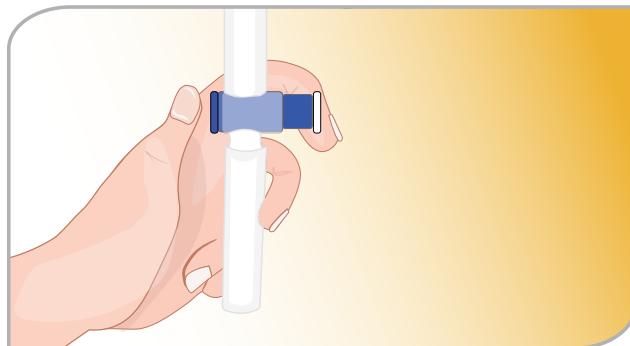

- 4.** Once the bag is empty, repeat the above step but in the opposite direction to close the tap.



## PRECAUTIONS

- Ensure the Urine Bag is never raised higher than the level of the bladder.
- Ensure the Urine Bag tubing is free from kinks at all times when connected to the catheter.
- At the completion of emptying the bag, check to ensure the T-Tap is completely closed.
- Ensure aseptic technique is maintained when handling, connecting and disconnecting the Urine Bag from the urinary catheter.

## CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.

Always read the label and follow the Instructions for Use. Our guidance is not intended to substitute professional medical advice and does not constitute health advice. If you have concerns or questions regarding the use of this product, seek advice from a qualified healthcare professional.



m|devices®

(02)8718 2800  
info@mdevices.com.au  
mdevices.com.au