


# Leg Bag with T-Tap


## INSTRUCTIONS FOR USE





**1.** Perform hand hygiene.

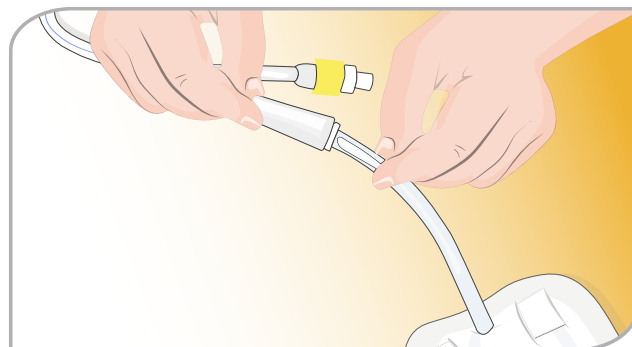


**2.** To disconnect or change your drainage bag, pinch the catheter above the drainage bag to stop the flow of urine and disconnect using a twisting motion.

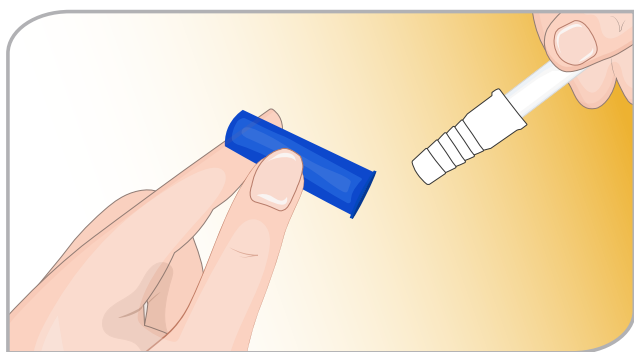

**3.** Remove blue protective cap from stepped connector on the inlet drainage tube and using aseptic technique, connect directly into the funnel end of the foley catheter. Ensure the connection is secure.

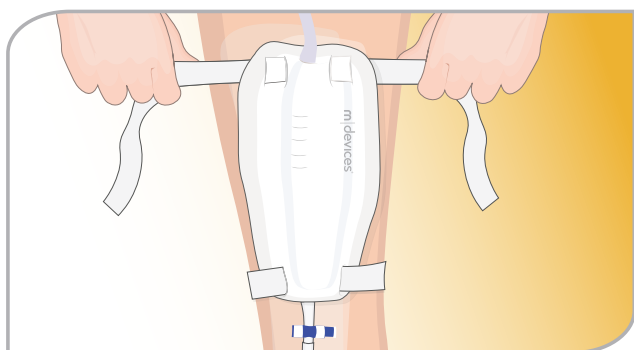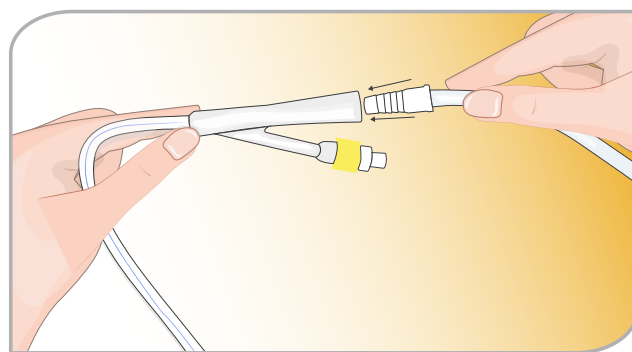


**4.** To secure your Leg Bag, wrap the two straps on the bag around the thigh, knee, or calf and secure at desired position using the adjustable elastic or silicone straps. Ensure there is some slack in catheter and room to move, so there is no pressure on urethra, however it is secure enough not to move vertically on your leg.

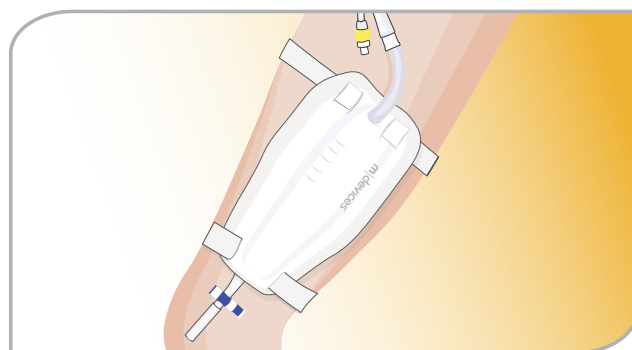


# Leg Bag with T-Tap

## INSTRUCTIONS FOR USE

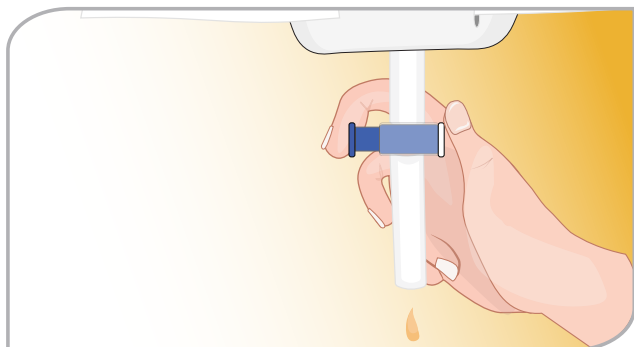

- 5.** To empty the bag (the T-Tap will be in a closed position when removed from the packaging), hold the T-Tap mechanism in your hand, place your thumb on the white capped button and slide across until it sits firmly against the T-piece. Urine will flow out.

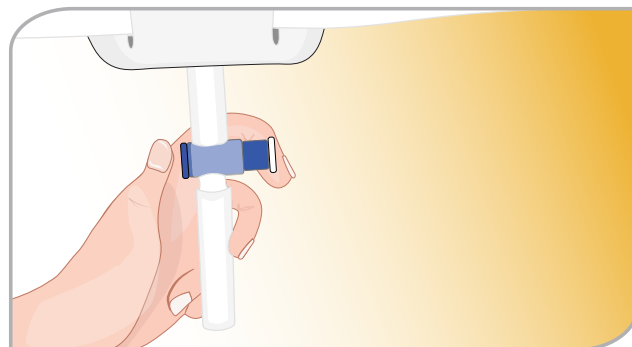

- 6.** Once empty, using the same action, slide the blue capped button back across until it sits firmly against the T-piece.



## CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.

Always read the label and follow the Instructions for Use. Our guidance is not intended to substitute professional medical advice and does not constitute health advice. If you have concerns or questions regarding the use of this product, seek advice from a qualified healthcare professional.



m|devices®

(02)8718 2800  
info@mdevices.com.au  
mdevices.com.au