




BALLOON REMOVAL PAXIS™

RAPID INSTILLATION KIT



Refer to the complete IFU for full Instructions for Use.

Rotate the white stopcock into the closed position.

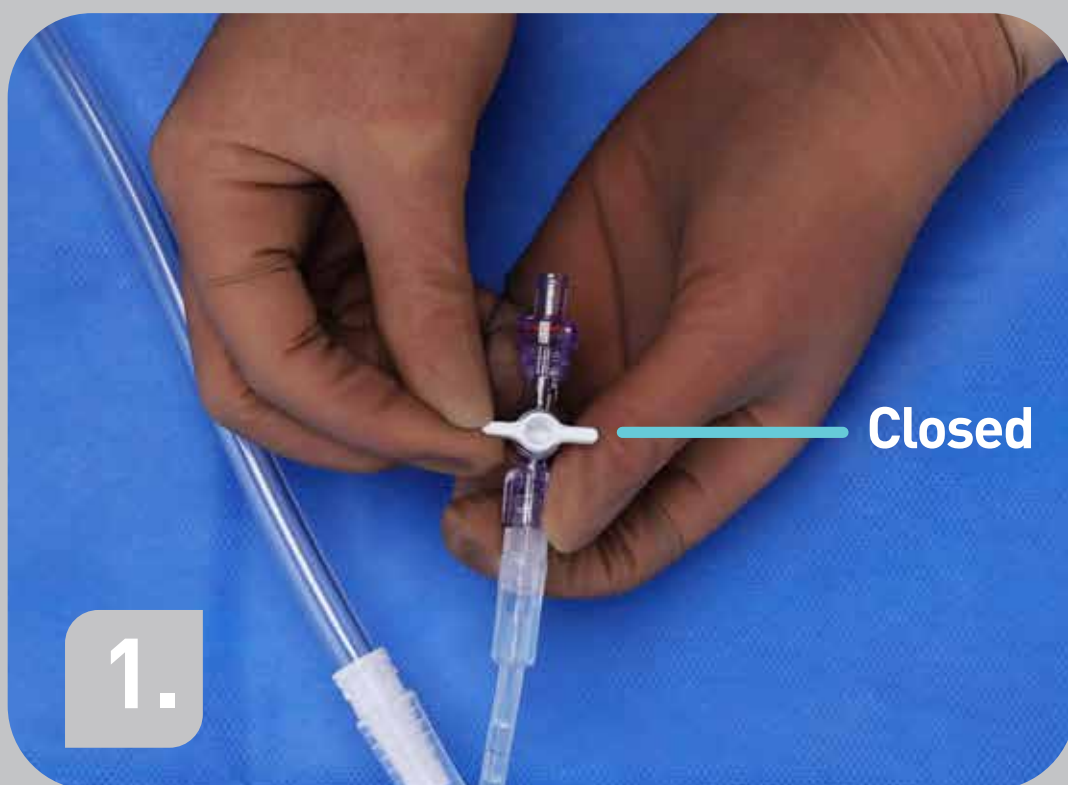

Detach the red back-check valve from the white stopcock




Attach the syringe ensuring connection is secure.



Rotate the white stopcock into the open position and aspirate fluid into the syringe.



To empty the fluid from the syringe, rotate the white stopcock into closed position, detach and empty the syringe, then reattach the syringe and rotate the stopcock into the open position. Repeat until deflated.

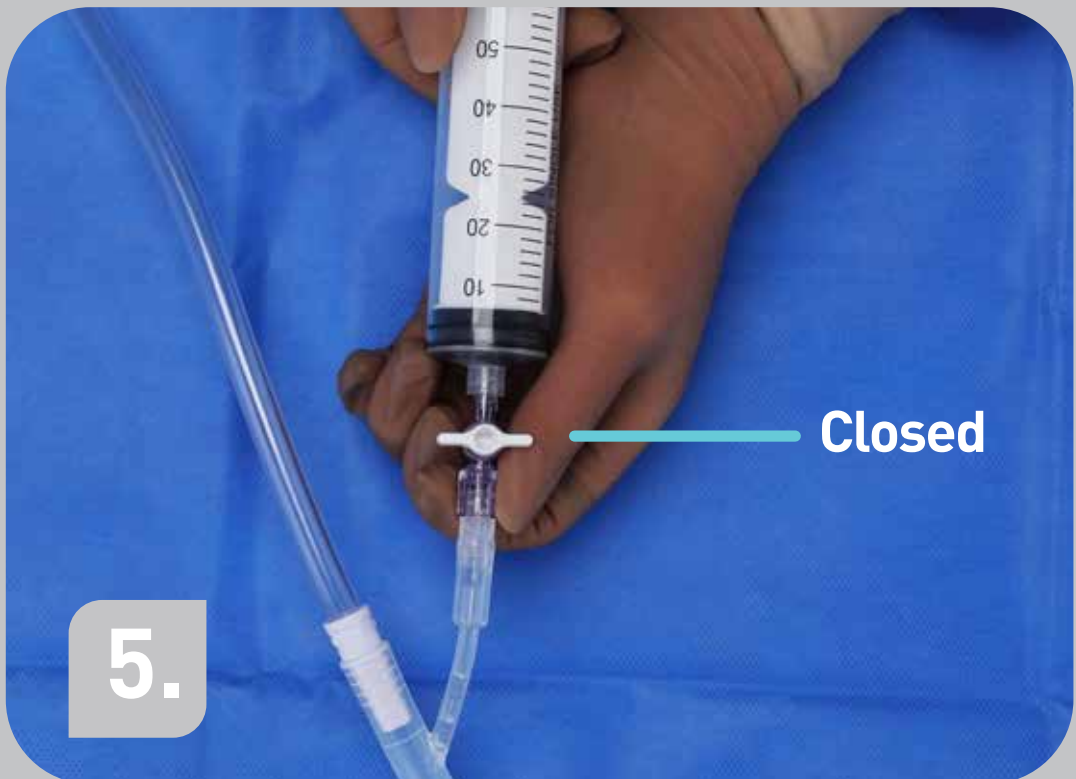

In an emergency, the catheter shaft may be cut to facilitate a more rapid deflation or the white stopcock can be turned to the open position to allow free flow deflation.



6.

Gently retract the deflated balloon and monitor the patient for signs of bleeding.

