



AN010000 - AN010001

Disposable Laryngoscope

Yeescope™

INSTRUCTION FOR USE

DESCRIPTION

m|devices Yeescope™ is a one-piece disposable laryngoscope with incorporated handle and blade. It has a light weight, patented angle design with a bright white light incorporated within the blade.

The Yeescope™ is available in MAC3 and MAC4 sizes.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON-STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Check the laryngoscope light before use.
2. To turn on the light, pull and completely remove the tab from the handle.
3. Use the right paraglossal approach to insert the blade and lift the laryngoscope handle by the hook to facilitate display of the larynx for intubation.
4. Dispose of Yeescope™ and batteries after use in accordance with hospital policy.

BATTERY REMOVAL

1. Using the red spanner provided in the box, attach the spanner over the hook on the end cap of the Yeescope™ handle.
2. Once the spanner is securely in position; using one hand hold the blade of the Yeescope™ firmly and with your other hand turn the spanner to unlock the end cap.
3. Remove end cap to release the batteries from the battery compartment. Dispose of the batteries as per facility protocol.
4. Dispose of the Yeescope™ as per Facility protocol.

PRECAUTIONS

- Only for use by experienced clinicians.
- This product contains 2 AA alkaline batteries.
- NOT for use in MR environment.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



 **Multigate Co., Ltd.**
7 Lanjiang Road 312000
Shaoxing China

 **RFX+CARE International A/S**
Bakkegaardsvej 408
3050 Humlebaek Denmark

