

AN010000 - AN010001 Disposable Laryngoscope Yeescop™

INSTRUCTION FOR USE

DESCRIPTION

m|devices Yeescop™ is a one-piece disposable laryngoscope with incorporated handle and blade. It has a light weight, patented angle design with a bright white light incorporated within the blade.

The Yeescop™ is available in MAC3 and MAC4 sizes.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON-STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Check the laryngoscope light before use.
2. To turn on the light, pull and completely remove the tab from the handle.
3. Use the right paraglossal approach to insert the blade and lift the laryngoscope handle by the hook to facilitate display of the larynx for intubation.
4. Dispose of Yeescop™ and batteries after use in accordance with hospital policy.

BATTERY REMOVAL

1. Using the red spanner provided in the box, attach the spanner over the hook on the end cap of the Yeescop™ handle.
2. Once the spanner is securely in position; using one hand hold the blade of the Yeescop™ firmly and with your other hand turn the spanner to unlock the end cap.
3. Remove end cap to release the batteries from the battery compartment. Dispose of the batteries as per facility protocol.
4. Dispose of the Yeescop™ as per Facility protocol.

PRECAUTIONS

- Only for use by experienced clinicians.
- This product contains 2 AA alkaline batteries.
- NOT for use in MR environment.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800



 **Multigate Co., Ltd.**
4/599 Wuxie Road
Yuecheng District, Shaoxing
Zhejiang, China 312000

 **RFX+CARE International A/S**
Gammel Strandvej 22B
2990 Nivå, Denmark

