

AN060000NS - AN066005NS Oxygen Masks

INSTRUCTION FOR USE

DESCRIPTION

m|devices Oxygen Masks are available with or without star lumen oxygen tubing and are offered in a range of four different sizes suiting paediatric to adult faces. All provide medium concentration and vary between elongated and standard mask shapes.

Oxygen tubing: 2.1m and 3m lengths.

The delivery of oxygen through m|devices Oxygen Mask enables the patient to be comfortable through the use of an adjustable strap and nose clip to ensure an adequate fit.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Remove the product from the packaging.
2. Connect tubing to mask.
3. Connect distal end of tubing to oxygen source.
4. Set oxygen source to prescribed flow rate, ensuring connections are secure and oxygen is flowing through the device.
5. Place the mask over the patient's nose and mouth with the elastic strap below the ears and around the neck.
6. Secure by gently pulling the ends of the elastic strap.
7. Mold the metal strip on the mask to fit the nose.

PRECAUTION

- Ensure all connections are secured and monitored during use.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800



 **Multigate Co., Ltd.**
4/599 Wuxie Road
Yuecheng District, Shaoxing
Zhejiang, China 312000

 **RFX+CARE International A/S**
Gammel Strandvej 22B
2990 Nivå, Denmark

