

m|devices®

AN070000NS - AN070005NS Nasal Oxygen Cannulas

INSTRUCTION FOR USE

DESCRIPTION

m|devices Nasal Oxygen Cannulas are designed to simplify oxygen administration and are soft and flexible to maximise patient comfort. Used to deliver oxygen when low or medium flow concentration is required and when the patient is in a stable state.

Available in infant, child and adult sizes and in two lengths: 2.1m and 3m oxygen tubing.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Remove the product from the packaging.
2. Attach distal end of tubing to the oxygen source.
3. Set the prescribed flow rate, ensuring all connections are secured and oxygen is flowing through the device.
4. Insert the nasal tips into the nostrils, passing the two plastic tubes over the ears and under the chin.
5. Gently adjust the plastic slide to secure the cannula to the nostrils.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801

 AIRWAY
MANAGEMENT Nasal Oxygen Cannulas

 **Multigate Co., Ltd.**
7 Lanjiang Road 312000
Shaoxing China

 **RFX+CARE International A/S**
Bakkegaardsvej 408
3050 Humlebaek Denmark

