



AN090000NS - AN090008NS Guedel Airways



INSTRUCTION FOR USE

DESCRIPTION

m|devices Guedel Airways are sizing colour coded and non-sterile.

Designed to provide additional support to the airway.

Wide sizing range starting from No. 000 through to No.6.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Select appropriate size by measuring from the center of the patient's mouth/incisors to the angle of the jaw.
2. Open package and remove Guedel.
3. Lubricate the distal end of the Guedel Airway if required, prior to insertion.
4. Ensure the inner surface curve is facing up towards the roof of the mouth.
5. Once the Guedel is one third inside the mouth, rotate the guedel 180 degrees over the tongue.
6. Gently advance the Guedel until the flange of the guedel is positioned against the lips.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800



AIRWAY MANAGEMENT Laryngeal Mask Airway



Multigate Co., Ltd.
4/599 Wuxie Road
Yuecheng District, Shaoxing
Zhejiang, China 312000



RFX+CARE International A/S
Gammel Strandvej 22B
2990 Nivå, Denmark



Rev.04