

m|devices®

## AN13000NS - AN135001NS Oxygen Tubing

### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices star lumen Oxygen Tubing design features a channeled inner lumen to resist kinks and occlusion, for the delivery of oxygen to the patient.

The tubing is attached to either a face mask, nasal cannula or nebuliser set.

Available in 4 different lengths; 2.1m, 3m, 4m and 30m.  
Optional stepped connector for the 3m length AN135001NS.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

#### CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

1. Remove the product from the packaging.
2. Attach the Oxygen Tubing to the oxygen source and set to the prescribed flow.
3. Check for oxygen flow through the tubing.
4. Attach to desired mask, nasal cannula or nebuliser, ensuring that all connections are secured.

m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801

 AIRWAY  
MANAGEMENT Oxygen Tubing

 **Multigate Co., Ltd.**  
7 Lanjiang Road 312000  
Shaoxing China

 **RFX+CARE International A/S**  
Bakkegaardsvej 408  
3050 Humlebaek Denmark

