



## AN131030NS

### Bubble Oxygen Tubing



#### INSTRUCTION FOR USE

##### DESCRIPTION

m|devices Bubble Oxygen Tubing has incremental sections of differing dimensions throughout the tubing length of 4mm-8mm ID. The total tubing length is 30m.

This tubing is designed for the clinician to cut the desired length from the 30m roll at the required tubing diameter section to suit the oxygen mask connection for a secure fit. The other end of the cut tubing will connect to the oxygen source connection.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**



##### CAUTION

- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.
- DO NOT re-use or re-sterilise

##### STEPS

1. Remove desired length of tubing from the box dispenser and using scissors, cut the tubing at the closest segment connection point to the required length.
2. Connect one end of the tubing to the oxygen source and set to the prescribed flow rate.
3. Connect the other end of the tubing to the desired product of choice.
4. Ensure all connections are secure.

##### m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801

m|devices®