

m|devices®

AN131030NS Bubble Oxygen Tubing

INSTRUCTION FOR USE

DESCRIPTION

m|devices Bubble Oxygen Tubing has incremental sections of differing dimensions throughout the tubing length of 4mm-8mm ID. The total tubing length is 30m.

This tubing is designed for the clinician to cut the desired length from the 30m roll at the required tubing diameter section to suit the oxygen mask connection for a secure fit. The other end of the cut tubing will connect to the oxygen source connection.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Remove desired length of tubing from the box dispenser and using scissors, cut the tubing at the closest segment connection point to the required length.
2. Connect one end of the tubing to the oxygen source and set to the prescribed flow rate.
3. Connect the other end of the tubing to the desired product of choice.
4. Ensure all connections are secure.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801

 AIRWAY
MANAGEMENT Oxygen Tubing

 **Multigate Co., Ltd.**
7 Lanjiang Road 312000
Shaoxing China

 **RFX+CARE International A/S**
Bakkegaardsvej 408
3050 Humlebaek Denmark

