



AN145001NS - AN145004NS Low Profile Anaesthetic Masks

INSTRUCTION FOR USE

DESCRIPTION

m|devices Low Profile Anaesthetic Masks are ergonomically shaped with a wide cuff that contours to the patient's face to obtain an effective seal.

The cushion-less design requires no inflation/deflation of the cushion prior to use as often required with traditional cushioned masks.

The masks are available in 4 sizes: Paediatric, Child, Small Adult and Large Adult and are colour coded for easy size identification.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use.
- DO NOT re-sterilise
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- NON-STERILE, DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacture and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Assess patient's face shape and size and select the appropriately sized Low Profile Anaesthetic Mask. Apply the mask to the face to check sizing if required.
2. Connect mask to breathing circuit or resuscitation device.
3. Ensure connections are firm.
4. Apply the mask to the patients face with gentle but firm pressure to create an effective mask seal.
5. After use, dispose of as per Facility protocol. The below is only a guide and clinician assessment will take precedence.

The below is only a guide and clinician assessment will take precedence.

SUGGESTED SIZING GUIDE

Code	Colour	Age	Weight (kg)	Connector Size
AN1450001NS	Pink	≤1 yr	≤10	15mm
AN1450002NS	Green	1-8 yrs	10-26	22mm
AN1450003NS	Purple	≤8 yrs	26-120	22mm
AN1450004NS	Grey	≤16 yrs	≤58	22mm

PRECAUTION

- Ensure the connection between the mask and the breathing circuit or resuscitation device is secure at all times.

m|devices®

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



Multigate Co., Ltd.
7 Lanjiang Road 312000
Shaoxing China



RFX+CARE International A/S
Bakkegaardsvej 408
3050 Humlebaek Denmark

