

HR2452-S

Swabable Needleless Access Site

i INSTRUCTION FOR USE

DESCRIPTION

m|devices Swabable Needleless Access Site has a split septum and lipid resistant valve offering high flow rates and can be accessed up to 300 times for both injection and aspiration.

It is luer activated and designed for use with luer lock and luer slip connectors. No other device is required for activation and a small twist is all that is required to ensure a firm connection.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

⚠ CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Using aseptic technique, open the packaging and remove the Swabable Needleless Access Site.
2. According to facility protocol, using a syringe filled with sterile fluid prime the valve to dispel air.
3. Remove the end protective cap and attach to the corresponding luer connection/catheter hub.
4. Disinfect the surface of the Swabable Needleless Access Site with an appropriate antiseptic swab as per facility protocol for at least 15-30 seconds, allow to completely dry.
5. Holding the Swabable Needleless Access Site, insert the luer connector with a straight motion, push and twist the luer ¼ turn to the right to lock in place. Ensure the connection is secure.
6. Once the medication/fluid has been injected, clamp the catheter at the patient end prior to disconnecting the syringe/connection from the Swabable Needleless Access Site. Facility protocol should be followed for the clamping sequence is different to the above suggestion.
7. To disconnect hold the Swabable Needleless Access Site firmly and with a twisting motion, turn the luer ¼ turn to the left to unlock and pull out the luer connector.
8. Flush the Swabable Needleless Access Site and connecting catheter with a syringe filled with sterile fluid after every use as per facility protocol.
9. Repeat above steps (4-8) for every instance the Swabable Needleless Access Site is accessed.

PRECAUTIONS

- Only luer connectors are recommended when accessing the Swabable Needleless Access Site.
- No sharp needles/instruments are to be used with Swabable Needleless Access Site.
- Ensure all connections are secure and monitored during use.
- When disconnecting, ensure the Swabable Needleless Access Site is held firmly and not held below on the catheter hub, as the Swabable Needleless Access Site could accidentally become disconnected when twisting the corresponding luer connector.
- Ensure aseptic technique is maintained at all times when accessing the Swabable Needleless Access Site and corresponding luer connectors/catheter hubs. Once the Swabable Needleless Access Site has been disinfected, it should not be allowed to touch any non-sterile surfaces.

Rev.03

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800



Multigate Co., Ltd.
4/599 Wuxie Road
Yuecheng District, Shaoxing
Zhejiang, China 312000

RFX+CARE International A/S
Gammel Strandvej 22B
2990 Nivå, Denmark

