

IR010000 & IR010002 Arthroscopy Irrigation Set

i INSTRUCTION FOR USE

DESCRIPTION

m|devices Arthroscopy Irrigation Sets have double spike perforators and are available with a stepped connector and silicone adaptor or with a male luer lock connector.

Designed for arthroscopy procedures to provide irrigation to the surgical cavity.

IR010000 is 265cm in length and IR010002 is 260cm - both measurements include 220cm tube length.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

! CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacture and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Close click clamps.
2. Hang fluid bags and remove protective cover from bag spike outlets.
3. Remove protective caps from spikes of the irrigation set and insert into fluid bag using aseptic technique.
4. To prime the irrigation line, invert flexible pump chamber and open the click clamps until the fluid flows out through the silicone tubing adaptor/male luer lock connector, checking that all air has been expelled.
5. Close click clamps.
6. Connect and secure by attaching the silicone tubing adaptor/male luer lock connector to the inline port on the arthroscope/ instrumentation sheath.
7. Control flow rate using the click clamps until desired flow rate is achieved.
8. Squeezing the flexible pump chamber will provide a push of fluid to flush the irrigation set if required to assist in clearing the surgical view.

PRECAUTION

- Use of the flexible pump chamber should only be performed by a qualified clinician as incorrect use could potentially cause high pressures within the surgical cavity.

SYMBOLS EXPLANATION (for reference only)

	Cautions
	See instructions for use
	Product reference/Art. No.
	For single use only
	Do not re-sterilise
	Do not use if packaging or product has been damaged or contaminated
	Batch number
	Date of manufacturing
	Use by/Expiry date
	Sterilised by Ethylene Oxide
	Single sterile barrier system
	European Authorised Representative
	Manufactured by
	Storage Condition
	Keep away from sunlight
	Keep dry
	Medical device

m|devices
27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800

S SURGICAL & SPECIALTIES Irrigation Sets

POLY MEDICURE LTD.
Plot No. 104-105, Sector 59
HSIIDC Industrial Area
Ballabhgarh, Faridabad
Haryana 121004, INDIA

Obetis S.a.
Boulevard Général Wahis 53
1030, Brussels
BELGIUM.

0123