



# IV010003

## Gravity Infusion Set with Needleless Access Site



### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices gravity infusion set with Needleless Access Site is used to administer solution from a bag to the patient's vascular system through an intravenous catheter inserted into a vein. This set offers a 15micron filter vented chamber and Robertsite and consists of 220cm in length.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**



**CAUTION**  
DO NOT use if the package or product has been damaged or contaminated.  
DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

#### STEPS:

Use aseptic technique

1. Close the roller regulator.
2. Remove protective cap from spike and insert into fluid bag/bottle (If using a rigid container, open vent on spike).
3. Suspend fluid bag/bottle and squeeze lower half of the drip chamber until half full.
5. To prime line, open the roller regulator and invert the needleless access site and allow fluid to flow through the tubing until all air is expelled from the line to male luer connector.
6. Close the roller regulator.
7. Remove protective cap from male luer lock connector.
8. Connect and secure by inserting male luer lock connector into access device and engaging rotating collar.
9. Control flow rate with roller regulator.

#### USING THE NEEDLELESS ACCESS SITE:

1. Swab access site with antiseptic before use.
2. Connect a luer slip or luer lock connector to the access site by pushing straight into the access site, whilst using a clockwise rotating motion.
3. When disconnecting, always hold the housing of the needleless access site to prevent disconnection from the line.
4. Repeat above steps each time the needleless access site is accessed.

#### PRECAUTIONS:

- The security of all connections should be checked once the circuit is established and monitored during use to prevent disconnection

1800 023 321

m|devices

27 Llewellyn Avenue

Villawood NSW 2163 Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

[mdevices.com.au](http://mdevices.com.au)

**m|devices®**