

m|devices®

IV030008

Transfusion Pump Set

with Luer Break



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices Transfusion Pump Set with Luer Break is designed for use to deliver blood, blood products and fluid with gravity from the bag/bottle into the patient's vascular system via a catheter. The set incorporates double spike perforators, hand pump that can be used to manually pump/-push fluid into the patient at the discretion of the user, often when blood/fluid needs to be delivered quickly or at interval speeds. In addition, a Needleless Access Site, non-return valve with a 300cm tubing length. The Luer break offers a point for detachment.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacture and the competent authority of the Member State in which the user and /or patient is established.

### STEPS

1. Close all three roller regulators.
2. Remove protective cap from spikes and insert into fluid containers (if using a rigid container, open vent on spike).
3. Suspend containers and open the two roller regulators below the perforators to allow fluid to flow to drip chamber.
4. Squeeze lower half of the drip chamber until half full.
5. To prime line, open the roller regulator below the drip chamber and invert both the trauma pump and needleless access site until all air is expelled from the line to the luer connector.
6. Close roller regulator.
7. Remove protective cap from male luer lock connector.
8. Connect and secure by inserting male luer lock connector into access device and engaging rotating collar.
9. Control flow rate with roller regulator.

### USING THE NEEDLELESS ACCESS SITE

1. Swab access site with antiseptic before use.
2. Connect a luer slip or luer lock connector to the access site by pushing straight into the access site, whilst using a clockwise rotating motion.
3. When disconnecting, always hold the housing of the Needleless Access Site to prevent disconnection from the line.
4. Repeat above steps each time the Needleless Access Site is accessed.

### PRECAUTIONS

- Do not use a needle or blunt cannula with the Needleless Access Site.
- The security of all connections should be checked once the circuit is established and monitored during use to prevent disconnection.
- When using the luer break to disconnect the distal portion of the set, ensure all connections are secure when reconnecting.

m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801



VENOUS ACCESS Blood Transfusion Sets



Multigate Co., Ltd.  
7 Lanjiang Road 312000  
Shaoxing China



RFX+Care International A/S  
Bakkegaardsvej 408  
3050 Humlebaek  
Denmark

