

m|devices®

IV050015

Microbore Extension Set

15cm with NeutralSite™

Male Luer Lock with Rotating Collar
and Removable Slide Clamp

 INSTRUCTION FOR USE

DESCRIPTION

m|devices IV Microbore Extension Set, single lumen with NeutralSite™ and removable slide clamp is 15cm in length. This set has a male luer lock with rotating collar at one end and the NeutralSite™ at the other end. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- The integrity of the NeutralSite™ should be confirmed by the clinician before and immediately after each use.
- Glass syringes are NOT compatible with the NeutralSite™.
- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

Ensure aseptic technique is followed at all times.

1. To prime, connect a pre-filled syringe or IV administration set with sterile fluid to the NeutralSite™. Ensure the click clamp is open.
2. To access NeutralSite™, insert syringe with a straight motion, push and twist using a clockwise motion.
3. Prime fluid slowly through the set until the fluid drips out of the distal end of the extension set. If air bubbles are still visible, continue flushing until completely removed.
4. Close click clamp until ready to use the set.
5. Remove protective cap from male luer connector and connect securely into access device or peripheral catheter and engage rotating collar.

USING THE NEUTRALSITE™

1. Using your regulatory approved disinfectant, thoroughly disinfect the entire surface of the NeutralSite™ using mechanical friction, for at least 30 seconds. Allow the NeutralSite™ to completely dry.
2. Holding the NeutralSite™, insert the luer lock syringe with a straight motion, push and twist to the right until locked. If using a luer slip syringe, insert with a straight motion and twist 1/4 turn to the right.
3. To disconnect; hold the NeutralSite™ firmly and with a twisting motion, turn the luer to the left to unlock and pull out the luer connector.
4. Flush the NeutralSite™ with a syringe filled with N/Saline after every use or as per facility protocol.
5. Repeat the above steps for every instance the NeutralSite™ is accessed.


PRECAUTIONS

- Do not use a needle or blunt cannula with NeutralSite™.
- The security of all connections should be checked before and during use to prevent disconnection.
- Clean the surface of the NeutralSite™ with an approved disinfectant as per facility protocol for at least 30 seconds, before and immediately after each use to remove any residual medication/blood from the surface.
- Flush the NeutralSite™ after every use or as per facility protocol.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



 **BQ Plus Medical Co., Ltd.**
No. 18, Che Ye Road
Che Dun Town, Songjiang
201611 Shanghai, China

 **Prolinx GmbH**
Brehmstr. 56
40239 Duesseldorf
Germany

