



IV051006

Minimum Volume Extension Set
20cm
Needleless Access Site and NRV



INSTRUCTION FOR USE

DESCRIPTION

m|devices IV Minimum Volume Extension Set with Needleless Access Site, non-return valve and is 20cm in length. This set is lipid resistant and has a female luer lock to male luer lock with rotating collar. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use
- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

STEPS

Ensure aseptic technique is followed at all times

1. Remove protective cap from female luer connector.
2. Connect a pre-filled syringe or IV administration set with sterile fluid to female luer connector.
3. To prime line using a syringe, invert the Needleless Access Site and depress plunger to flush fluid through the set and until the fluid drips out of the distal end of the extension set. Check to ensure that all air from extension set tubing has been expelled. If air bubbles are still visible, continue flushing until completely removed.
4. To prime line by connecting to an already existing primed IV administration set, open roller clamp on IV administration set, invert Needleless Access Site and allow fluid to flow through extension set tubing until it drips out of the distal end of the set. Check to ensure that all air from extension set tubing has been expelled. If air bubbles are still visible, continue flushing until completely removed. Close roller clamp.
5. Remove protective cap from male luer connector.
6. Connect and secure by inserting male luer connector into access device or peripheral catheter and engage rotating collar.

USING THE NEEDLELESS ACCESS SITE

- Swab access site with antiseptic before use.
- Connect a luer slip or luer lock connector to the access site by pushing straight into the access site, whilst using a clockwise rotating motion.
- Repeat above steps each time the Needleless Access Site is accessed.

PRECAUTION

- The security of all connections should be checked once established and monitored during use to prevent disconnection.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163

Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

m|devices[®]