



IV052100

Standard Bore Extension Set
495cm
Injection Site

INSTRUCTION FOR USE

DESCRIPTION

m|devices Standard Bore Extension set has a standard injection site and is 495cm in length. This set has a female luer lock to male luer lock with rotating collar. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

STEPS

Ensure aseptic technique is followed at all times

1. Remove protective cap from female luer connector.
2. Connect a pre-filled syringe or IV administration set with sterile fluid to female luer connector.
3. To prime line using a syringe, depress plunger to flush fluid through the set whilst inverting the injection site and until the fluid drips out of the distal end of the extension set. Check to ensure that all air from extension set tubing and injection site has been expelled. If air bubbles are still visible, continue flushing until completely removed.
4. To prime line by connecting to an already existing primed IV administration set, open roller clamp on IV administration set whilst inverting the injection site to allow fluid to flow through extension set tubing until it drips out of the distal end of the set. Check to ensure that all air from extension set tubing and injection site has been expelled. If air bubbles are still visible, continue flushing until completely removed. Close roller clamp.
5. Remove protective cap from male luer connector.
6. Connect and secure by inserting male luer connector into access device or peripheral catheter and engage rotating collar.

PRECAUTIONS

- The security of all connections should be checked once established and monitored during use to prevent disconnection.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

m|devices[®]