

m|devices®

IV052300

Multi Lumen Standard Bore
Extension Set

15cm
2 Lumens

 INSTRUCTION FOR USE

DESCRIPTION

m|devices Multi Lumen Standard Bore Extension Set with twin lumens is 15cm in length. This set has two slide clamps and a female luer lock to male luer lock connector with rotating collar. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

Ensure aseptic technique is followed at all times

1. Remove protective cap from female luer connector.
2. Connect a prefilled syringe or primed IV line with sterile fluid to female luer connector.
3. To prime one lumen at a time; close one of the two slide clamps and allow fluid to flow through the open lumen to the male luer connector to expel all air. Close the slide clamp on primed lumen and repeat this step to prime the remaining other lumen until all air has been removed from the set.
4. Remove protective cap from male luer connector.
5. Connect and secure by inserting male luer connector into access device and engage rotating collar.


PRECAUTION


- The security of all connections should be checked once established and monitored during use to prevent disconnection.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801

 VENOUS
ACCESS Extension Sets

 **BQ Plus Medical Co., Ltd.**
No. 18, Che Ye Road
Che Dun Town, Songjiang
201611 Shanghai, China

 **ProInx GmbH**
Brehmstr. 56
40239 Duesseldorf
Germany

STERILE EO   MD 

   CE 0197