



# IV052400

Multi Lumen Standard  
Bore Extension Set 15cm  
3 Lumens



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices Multi Lumen Standard Bore Extension Set with triple Lumens, is 15cm in length. This set has three slide clamps and a female luer lock to male luer lock connector with rotating collar. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**



### CAUTION

- Single Use
- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

### STEPS

Ensure aseptic technique is followed at all times

1. Remove protective cap from female luer connector.
2. Connect a prefilled syringe or primed IV line with sterile fluid to female luer connector.
3. To prime one lumen at a time; close one of the three slide clamps and allow fluid to flow through the open lumen to the male luer connector to expel all air.
4. Close the slide clamp on primed lumen and repeat this step to prime the two remaining lumens until all air has been removed from the set.
4. Remove protective cap from male luer connector.
5. Connect and secure by inserting male luer connector into access device and engage rotating collar.

### PRECAUTION

- The security of all connections should be checked once established and monitored during use to prevent disconnection.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices®**