



# IV053001

Microbore Extension Set 15cm  
2 Lumens  
with Needleless Access Site



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices IV Microbore Extension Set, 2 Lumen, is 15cm in length. This set has a Needleless Access Site on the distal ends of both lumens both with slide clamps and a male luer lock connector with rotating collar. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

### CAUTION

- Single use.
- DO NOT use a needle or blunt cannula with Needleless Access Site.
- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

### STEPS

Ensure aseptic technique is followed at all times

1. Remove protective cap from female luer connector.
2. Connect a prefilled syringe or primed IV line with sterile fluid to female luer connector.
3. To prime one lumen at a time; close one of the two slide clamps and allow fluid to flow through the open lumen to the Needleless Access Site to expel all air from the lumen.
4. Close the slide clamp on the primed lumen and repeat this step to prime the other remaining lumen, until all air has been removed from the set.

### USING THE NEEDLELESS ACCESS SITE

- Swab access site with antiseptic before use.
- Connect a luer slip or luer lock connector to the access site by pushing straight into the access site, whilst using a clockwise rotating motion.
- Repeat above steps each time the Needleless Access Site is accessed.

### PRECAUTION

- The security of all connections should be checked once established and monitored during use to prevent disconnection.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163

Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices<sup>®</sup>**