



# IV070000

## Vented Transfer Set


### INSTRUCTION FOR USE

#### DESCRIPTION:

The m|devices Transfer Set has a vented perforator and a click clamp at one end and a male luer lock at the other.

The inner tubing diameter is 3mm and the total length is 20cm.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

 **CAUTION**  
DO NOT use if the package or product has been damaged or contaminated.  
DO NOT store at extreme temperatures and humidity, avoid direct sunlight.  
DO NOT re-use or re-sterilise.

#### STEPS:

1. Remove product from packaging, maintaining aseptic technique and close click clamps.
2. Remove protective cap from the spike and insert into fluid bag/container. In the case of a rigid container, open the vent on the spike.
3. Suspend the bag/container.
4. To prime the line, open click clamp and allow fluid to flow through to the male luer connector, expelling all air from the line.
5. Close click clamp.
6. Connect and secure by inserting the male luer lock connector into the access device and engage rotating collar.
7. Control fluid rate with the click clamp.

#### PRECAUTION:

The security of all connections should be checked once the circuit is established and monitored during use to prevent disconnection.

**m|devices**  
27 Llewellyn Avenue  
Villawood NSW 2163 Australia  
T +61 (0)2 8718 2900  
F +61 (0)2 8718 2801  
mdevices.com.au

**m|devices<sup>®</sup>**