



IV071000

Double Spike Adaptor



INSTRUCTION FOR USE

DESCRIPTION:

The m|devices Double Spike Adaptor is designed with two vented spikes, roller regulator and a septum connector.

The inner tubing diameter is 3mm and total length is 30cm.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.



CAUTION
DO NOT use if the package or product has been damaged or contaminated.
DO NOT store at extreme temperatures and humidity, avoid direct sunlight.
DO NOT re-use or re-sterilise.

STEPS:

1. Remove product from packaging, maintaining aseptic technique.
2. Close roller regulators on the double spike adaptor.
3. Remove protective cap from spikes and insert into fluid bag/containers.
4. Suspend bag/containers.
5. Remove protective cap from the in-line port and connect to corresponding IV administration set.
6. Close roller regulator on IV administration set.
7. Open roller regulators on the double spike adaptor to allow the fluid to prime to the in-line port through to the IV administration set.
8. To further prime past the in-line port, follow instructions to the administration set until all air has been expelled from the line to the luer adaptor.

PRECAUTION:

The security of all connections should be checked once the circuit is established and monitored during use to prevent disconnection.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163 Australia
T +61 (0)2 8718 2900
F +61 (0)2 8718 2801
mdevices.com.au

m|devices®