



# IV071000

## Double Spike Adaptor



### INSTRUCTION FOR USE

#### DESCRIPTION

The m|devices Double Spike Adaptor is designed with two spikes, two roller regulators and a septum connector.

The inner tubing diameter is 3mm and total length is 30cm.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

#### CAUTION

- Single use
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacture and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

1. Remove product from packaging, maintaining aseptic technique.
2. Close roller regulators on the double spike adaptor.
3. Remove protective cap from spikes and insert into fluid bag/containers.
4. Suspend bag/containers.
5. Close all roller regulators and/or click clamps on the administration set that will be connected to the double spike set.
6. Insert the administration set spike into the in-line port of the double spike adapter until the connection is secure.
7. Open roller regulators on the double spike adaptor to allow the fluid to prime to the inline port through to the IV administration set.
8. To further prime past the in-line port, follow instructions to the administration set until all air has been expelled from the line to the luer adaptor.

#### PRECAUTION

- The security of all connections should be checked once established and monitored during use to prevent disconnection.

# m|devices®

#### m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801



**BQ Plus Medical Co., Ltd**  
No. 18, Che Ye Road  
Che Dun Town, Songjiang  
201611 Shanghai, China



**Prolinx GmbH**  
Brehmstr. 56  
40239 Duesseldorf  
Germany

