



IV071001

Vented Double Spike Adaptor
30cm

INSTRUCTION FOR USE

DESCRIPTION

The m|devices Double Spike Adaptor is designed with two vented spikes, roller regulators and a septum connector.

The inner tubing diameter is 3mm and total length is 30cm

FOR USE BY AN ADEQUATELY TRAINED USER. THE BELOW IS ONLY A SUGGESTION AND THE USER SHOULD BE THOROUGHLY EDUCATED PRIOR TO ATTEMPTING TO SELF ADMINISTER INSULIN

CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Remove product from packaging, maintaining aseptic technique.
2. Close roller regulators on the Double Spike Adaptor.
3. Remove protective cap from spikes and insert into fluid bag/containers.
4. Suspend bag/containers.
5. Insert spike of chosen IV administration set into the septum. Ensure the spike pierces the septum.
6. Open roller regulators on the Double Spike Adaptor to allow the fluid to prime to the inline port through to the IV administration set. Open vents if required to improve flow.
7. To further prime past the in-line port, follow instructions to the administration set until all air has been expelled from the line to the luer adaptor.

PRECAUTION

- The security of all connections should be checked once the set is established and monitored during use to prevent disconnection.


m|devices®

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801

 **Extension Sets**

 **BQ Plus Medical Co., Ltd.**
No. 18, Che Ye Road
Che Dun Town, Songjiang
201611 Shanghai, China

 **Prolinx GmbH**
Brehmstr. 56
40239 Duesseldorf
Germany

STERILE EO    **MD**

   **CE 0197**