

# IV071001

## Vented Double Spike Adaptor

30cm

### INSTRUCTION FOR USE

#### DESCRIPTION

The m|devices Double Spike Adaptor is designed with two vented spikes, roller regulators and a septum connector.

The inner tubing diameter is 3mm and total length is 30cm

**FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

#### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

1. Remove product from packaging, maintaining aseptic technique.
2. Close roller regulators on the Double Spike Adaptor.
3. Remove protective cap from spikes and insert into fluid bag/containers.
4. Suspend bag/containers.
5. Insert spike of chosen IV administration set into the septum. Ensure the spike pierces the septum.
6. Open roller regulators on the Double Spike Adaptor to allow the fluid to prime to the in-line port through to the IV administration set. Open vents if required to improve flow.
7. To further prime past the in-line port, follow instructions to the administration set until all air has been expelled from the line to the luer adaptor.

#### PRECAUTION

- The security of all connections should be checked once the set is established and monitored during use to prevent disconnection.

#### m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61 (0)2 8718 2800



 **BQ Plus Medical Co., Ltd.**  
No. 18, Che Ye Road  
Che Dun Town, Songjiang  
201611 Shanghai, China

 **Prolinx GmbH**  
Brehmstr. 56  
40239 Duesseldorf  
Germany

