



IV140431- IV141330

Insulin Pen Needles



INSTRUCTION FOR USE

DESCRIPTION

m|devices Insulin Pen Needles consist of; a needle holder, needle tube, tip sheath, and needle seat sheath.

Available in a wide range of gauges and lengths to meet all clinical requirements.

This product can be used with the following brands of insulin pens: Novopen, Wanbangpen, Prandilin, HumaPen, Gansulinpen and KlikSTAR.

FOR USE BY AN ADEQUATELY TRAINED USER. THE BELOW IS ONLY A SUGGESTION AND THE USER SHOULD BE THOROUGHLY EDUCATED PRIOR TO ATTEMPTING TO SELF ADMINISTER INSULIN.



CAUTION

Single use
DO NOT re-sterilise
DO NOT use if packaging or product has been damaged or contaminated.
DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

STEPS:

1. Tear open the cover on the needle holder
2. Align the needle end of the needle holder directly with the refill and screw it on the pen
3. Remove the outer needle cap and the inner needle cap. When removing the inner needle cap, avoid bending the needle and remove the inner needle cap.
4. With the pen and needle facing up, gently flick the pen with your finger, until the bubble floats to the top. Press the injection button until the needle tip has visible insulin discharged
5. Select injection site and using an approved disinfection solution/swab prepare injection location. Allow to completely dry.
6. Directly enter the needle vertically into the skin.
7. Press the injection button for 5-10 seconds or until the full insulin dose has been administered.
8. Pull out the outer needle cap of the needle cover, unscrew the needle and dispose into a sharps container.

PRECAUTIONS:

- When using the super short and slim 32G (0.25) insulin needles, injecting vertically is achievable and there is no need for holding/pinching the skin. When using other specifications of insulin needles, the needle should be injected into the skin at the angle of 90° or 45° as per instructions.
- At each injection location (i.e. abdomen), assure that the location of the injection site for each use is alternated to reduce the risk of adipose hyperplasia. As a guide, the distance should be approximately 3cm between each injection site.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163 Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801
mdevices.com.au

m|devices®