



## IV140431-IV141330 Insulin Pen Needles

### INSTRUCTION FOR USE

#### DESCRIPTION

Intended Use: m|devices Insulin Pen Needles consist of; a needle holder, needle tube, tip sheath, and needle seat sheath.

Available in a wide range of gauges and lengths to meet all clinical requirements.

This product can be used with the following brands of insulin pens: Novopen, Wanbangpen, Prandilin, HumaPen, Gansulinpen and KlikSTAR.

**FOR USE BY AN ADEQUATELY TRAINED USER. THE BELOW IS ONLY A SUGGESTION AND THE USER SHOULD BE THOROUGHLY EDUCATED PRIOR TO ATTEMPTING TO SELF ADMINISTER INSULIN.**

#### CAUTION

- Single Use only
- DO NOT re-sterilise
- DO NOT use if packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

#### STEPS

1. Select desired needle gauge and length as per health care professional recommendation.
2. Tear open the cover on the needle holder.
3. Align the needle end of the needle holder directly with the refill and screw it on the pen.
4. Remove the outer needle cap and the inner needle cap. When removing the inner needle cap, avoid bending the needle and remove the inner needle cap.
5. With the pen and needle facing up, gently flick the pen with your finger, until the bubble floats to the top. Press the injection button until the needle tip has visible insulin discharged.
6. Select injection site and using an approved disinfection solution/swab prepare injection location. Allow to completely dry.
7. Insert insulin pen needle as per recommendations by your health care professional.
8. Press the injection button for 5-10 seconds or until the full insulin dose has been administered.
9. Pull out the outer needle cap of the needle cover, unscrew the needle and dispose into a sharp's container.

#### PRECAUTION

- At each injection location (i.e. abdomen), assure that the location of the injection site for each use is alternated to reduce the risk of adipose hyperplasia. As a guide, the distance should be approximately 3cm between each injection site.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices<sup>®</sup>**