



# IV181001 – IV181060

Luer Lock Syringe  
without Needle



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices Luer Lock Syringes without Needle, have a plunger stopper with a double sealing ring. They are available in a broad range of sizes: 1mL, 3mL, 5mL, 10mL, 20mL, 30mL, 50mL and 60mL. They are free from latex and DEHP and are lipid resistant.

Commonly used to draw up measured medication/solution to be delivered to the patient.

**FOR USE BY A CERTIFIED CLINICIAN THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

### CAUTION

- Single use
- DO NOT re-use or re-sterilise
- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

### STEPS:

1. Select the preferred syringe size.
2. Using aseptic technique, peel packaging to remove the syringe from pouch.
3. Draw up the prescribed dosage of medication/ fluid (either using a selected drawing up hypodermic which is applied to the syringe or by securing the luer lock connection directly into the ampoule) as per facility protocol.
4. Ensure all air has been expelled from syringe.
5. Securely connect syringe to corresponding hypodermic needle or connection point.
6. When in desired location, push the plunger to deliver to deliver the required dose.
7. Dispose of syringe as per facility protocol.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801

**m|devices®**