



IV189001

Luer Lock Syringe 1mL

Low Dead Space

INSTRUCTION FOR USE

DESCRIPTION

m|devices 1mL Luer Lock Syringe is intended to deliver an accurate metered dosage throughout the administration of hypodermic injection. The Low Dead Space allows for minimal residual volume due to the displacement plunger tip. The high transparency barrel allows the syringe contents to be clearly seen and the finger grip on the plunger end ensures stability when administering contents.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Using aseptic technique, peel packaging to remove the syringe from the pouch.
2. Draw up the prescribed dosage of medication/fluid as per facility protocol (either using a selected hypodermic needle or by securing the luer lock connection directly into an ampoule).
3. Ensure all air has been expelled from the syringe prior to administration.
4. Securely connect syringe to corresponding hypodermic needle or connection point.
5. To administer syringe contents, gently push the plunger to the very end of the syringe barrel to deliver the prescribed dose.
6. Dispose of syringe as per facility protocol.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800



VENOUS
ACCESS

Sterile Syringes



**Jiangsu Jichun Medical
Devices Co., Ltd.**
No. 98 Baiyang Bridge
Zhenglu Town, Tianning
Changzhou 213111 Jiangsu
China



Caretechion GmbH
Niederrheinstr 71
40474 Duesseldorf
Germany



Rev.03