

m|devices®

SU102500 - SU103100

Insufflation Filter with Tubing Set

INSTRUCTION FOR USE

DESCRIPTION

m|devices Insufflation Filter with Tubing Sets ensure reliable and effective gas delivery for minimally invasive laparoscopic surgery.

Available in 3m length. Also available in a variety of connectors suitable for your insufflation unit.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacture and the competent authority of the Member State in which the user and /or patient is established.

STEPS


1. Peel open the packaging to access the sterile contents.
2. Place the components onto a sterile field.
3. Attach the connection fitting, which is located closest to the filter, to the output insufflation gas port.
4. Connect the opposite Luer connector to the Veress Needle or gas port of the Surgical Trocar Cannula.
5. Once connected, proceed with the insufflation process as per your surgical protocol.


PRECAUTIONS


- Ensure connection is secure before use.
- This device may only be used by suitably trained healthcare personnel.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61 (0)2 8718 2800

 **S** SURGICAL & SPECIALTIES Insufflation Filter with Tubing Set

 **Advanced Medical Design Co., Ltd.**
4F-5F, No29, Wuquan 5th Rd.
Wugu Dist., New Taipei City
24888 Taiwan R.O.C

 **OBELIS S.A**
Bd. Général Wahis, 53
1030 Brussels, Belgium

Rev01