

m|devices®

# UR019500

## Refill for Nelaton Catheter Starter Kit

### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices Refill for Nelaton Catheter Starter Kit is specifically designed with the user in mind, for a convenient travel option to insert into the m|devices Starter Kit Pouch.

Each Refill Kit contains one of each, necessary item required to support intermittent catheterisation:

- The hand cleansing gel sachet is ideal for pre-cleansing the hands prior to catheterising when hand washing facilities are not available.
- The NaCL 0.9% wet wipe can be used to cleanse the urethral area prior to catheterising and the dry hand towel can be used to dry the urethral area or hands if required.
- The non-transparent waste bag with adhesive strip, allows the bag to be secured to a desired location for discreet and convenient disposal of the wipes and catheter after the procedure.

#### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

##### Waste Bag

1. Unfold waste bag and peel off the adhesive strip backing and adhere to desired location (wall or toilet cubicle door).
2. Packaging and used items from the procedure are to be placed into the bag for disposal into an appropriate waste bin.

##### Hand Cleansing Gel

1. Tear open sachet from one of the top corners and peel packaging across the top to expose an opening.
2. Open one hand palm facing up and using the other hand to squeeze the sachet to allow the gel to flow out onto the exposed palm until all contents have been emptied from the sachet.
3. Dispose of the empty sachet into the waste bag.
4. Rub hands vigorously ensuring all surfaces of the hands and fingers come in contact with the gel. Rub until the gel has dried.

##### NaCL 0.9% Wet Wipe

1. Tear open packaging along top of packet to expose the wipe.
2. Remove wipe and completely unfold.
3. Wipe over the urethral area until area is cleansed.
4. Dispose of wipe into the waste bag.

##### Dry Hand Towel

1. Unfold hand towel prior to use.
2. After use, dispose of into the waste bag.

#### PRECAUTIONS

- For use by an adequately trained user. The instructions for use are only a suggestion and the user should be thoroughly educated prior to attempting to self catheterise and be aware of infection control principles.

m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801



CONTINENCE CARE Sterile Surgical Procedure Packs



Multigate Co., Ltd.  
7 Lanjiang Road 312000  
Shaoxing China



RFX+Care International A/S  
Bakkegaardsvej 408  
3050 Humlebaek  
Denmark

