

UR020001

Cystoscopy/TUR Irrigation Set



INSTRUCTION FOR USE

DESCRIPTION

m|devices Cystoscopy/TUR Irrigation Set is designed for cystoscopy or bladder irrigation following transurethral resection of the prostate or bladder tumor to support the removal of debris, clots and tissue.

Double spike perforation, 260cm tubing length and a stepped connector with silicone tube end supports the full range of clinical needs.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

⚠ CAUTION

- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.
- DO NOT re-use or re-sterilise

STEPS

1. Close click clamps.
2. Hang fluid bags and remove protective cover from bag spike outlets.
3. Remove protective caps from spikes of the Irrigation Set and insert into fluid bags using aseptic technique.
4. To prime the irrigation line, invert flexible chamber and open the click clamps until the fluid flows out through the silicone tubing adaptor, checking that all air has been expelled.
5. Close click clamps.
6. Connect and secure by attaching the silicone tubing adaptor to the inline port on the endoscope/instrumentation sheath.
7. Control flow rate using the click clamps until desired flow rate is achieved.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801

m|devices®