



# UR043001 - UR045005

## Hydrophilic Nelaton Catheters

### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices Hydrophilic Nelaton Catheters are available in a variety of lengths;

- Paediatric 30cm
- Female 18cm
- Male 40cm

The smooth water soluble polymer coating facilitates patient's use without the need for any lubricants. The heat polished closed distal end and side eyelets support a smooth and painless insertion and efficient drainage of urine.

**FOR USE BY AN ADEQUATELY TRAINED USER. THE BELOW IS ONLY A SUGGESTION AND THE USER SHOULD BE THOROUGHLY EDUCATED PRIOR TO ATTEMPTING TO SELF CATHETERISE AND BE AWARE OF INFECTION CONTROL PRINCIPLES.**

#### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

1. Peel package 2-4 cm and add sterile water to pack to sufficiently lubricate the entire catheter.
2. Wait 15-30 seconds to ensure hydrophilic coating has been activated.
3. Remove catheter from package taking care not to contaminate.
4. Insert the now lubricated catheter gently into the urethra passage until urine starts to flow. Urine can be drained into toilet or into a urine drainage bag connected to the catheters funnel.
5. When bladder is empty, slowly remove the catheter and place the used catheter into empty package for disposal.


#### PRECAUTION


- Maintaining aseptic technique during insertion is crucial. If the catheter becomes contaminated, discard and commence procedure using a new catheter.

# m|devices®

#### m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801

 **Multigate Co., Ltd.**  
7 Lanjiang Road 312000  
Shaoxing China

 **RFX+Care International A/S**  
Bakkegaardsvej 408  
3050 Humlebaek  
Denmark



0123