



## UR062000 – UR062008 Urine Bags

### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices Urine Bags are available with or without non-return valves and variations in tubing length ensure the bags are optimised for different settings.

The T-Tap end allows for ease of use to empty the Urine Bag and the clear incremental measurements allow for visual urine output.

**THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

#### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

#### STEPS

1. Perform hand hygiene.
2. Remove bag from the packaging and check that the T-Tap at the distal end of the bag is closed.
3. Using aseptic technique, remove cap from the drainage tubing and connect directly into the funnel of the urinary catheter.
4. Ensure the connection is secure.
5. Hang the bag at or below the patient's bladder level.

#### TO EMPTY THE URINE BAG

1. Perform hand hygiene and don gloves.
2. Place a receptacle directly under the T-Tap of the Urine Bag.
3. Holding the T-Tap in one hand, with fingers on either side of the tap, gently push against the mechanism to slide the tap across the T connection to open the tap to allow the urine to empty.
4. Once the bag is empty, repeat the above step but in the opposite direction to close the tap.

#### PRECAUTIONS

- Ensure the Urine Bag is never raised higher than the level of the bladder.
- Ensure the Urine Bag tubing is free from kinks at all times when connected to the catheter.
- At the completion of emptying the bag, check to ensure the T-Tap is completely closed.
- Ensure aseptic technique is maintained when handling, connecting and disconnecting the Urine Bag from the urinary catheter.

#### m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61(0)2 8718 2800  
F +61(0)2 8718 2801



 **Multigate Co., Ltd.**  
7 Lanjiang Road 312000  
Shaoxing China

 **RFX+Care International A/S**  
Bakkegaardsvej 408  
3050 Humlebaek  
Denmark

