

## UR062004 – UR062005

### Drainage Bags



#### INSTRUCTION FOR USE

#### DESCRIPTION

Intended Use: m|devices urine drainage bags without outlet are designed for the drainage of urine or other bodily fluids. Clear incremental markings provide precise volumes of drainage.

THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.



#### CAUTION

Single patient use  
DO NOT re-sterilise  
DO NOT use if packaging or product has been damaged or contaminated.  
DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

#### STEPS:

1. Perform hand hygiene.
2. Remove bag from the packaging.
3. Using aseptic technique, remove cap from the drainage tubing and connect directly into the funnel of the urinary catheter.
4. Ensure the connection is secure.
5. Hang the bag at or below the patient's bladder level.
6. Once bag is full, perform hand hygiene and don gloves.
7. Maintaining aseptic technique, disconnect the drainage bag from the urinary catheter and dispose of as per facility protocol.
8. Repeat the above procedure if re-connecting a new drainage bag.

#### PRECAUTIONS:

- Ensure the drainage bag is never raised higher than the level of the bladder
- Ensure the drainage bag tubing is free from kinks at all times when connected to the catheter
- Ensure aseptic technique is maintained when handling connecting and disconnecting the drainage bag from the urinary catheter.

1800 023 321

m|devices

27 Llewellyn Avenue

Villawood NSW 2163 Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices™**