



UR062004 – UR062005

Urine Bags

No Outlet

INSTRUCTION FOR USE

DESCRIPTION

m|devices Urine Bags without outlet are designed for the drainage of urine or other bodily fluids.

The bags have no drainage outlet and offer clear incremental marking to measure precise volumes. Both bags have non-return valves and 120cm tube length.

THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Perform hand hygiene.
2. Remove bag from the packaging.
3. Using aseptic technique, remove cap from the drainage tubing and connect directly into the funnel of the urinary catheter.
4. Ensure the connection is secure.
5. Hang the bag at or below the patient's bladder level.
6. Once bag is full, perform hand hygiene and don gloves.
7. Maintaining aseptic technique, disconnect the drainage bag from the urinary catheter and dispose of as per facility protocol.
8. Repeat the above procedure if reconnecting a new Urine Bag.

PRECAUTIONS

- Ensure the Urine Bag is never raised higher than the level of the bladder
- Ensure the Urine Bag tubing is free from kinks at all times when connected to the catheter
- Ensure aseptic technique is maintained when handling, connecting and disconnecting the Urine Bag from the urinary catheter.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



 **Multigate Co., Ltd.**
7 Lanjiang Road 312000
Shaoxing China

 **RFX+Care International A/S**
Bakkegaardsvej 408
3050 Humlebaek
Denmark

