

# UR064000

## Urine Bag 4000mL

T-Tap Non-Return Valve 150cm Tubing  
Integrated Hanger with Rope



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices Urine Bag has a large 4000mL volume capacity, non-return valve, 150cm tubing length with bonded stepped connector and integrated hanger with rope.

The urine bag is used to collect and contain urine via a urinary catheter or connected leg bag. This sterile bag is ideal for overnight drainage due to its large volume capacity. The T-Tap outlet allows for ease of use to empty the drainage bag and the clear incremental measurements allow for visual urine output.

THE BELOW IS ONLY A SUGGESTION AND FACILITY  
PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL  
PROCEDURES WHERE THIS PRODUCT IS USED.

### CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

### STEPS

1. Perform hand hygiene.
2. Remove bag from the packaging and check that the T-Tap at the distal end of the bag is closed.
3. Using aseptic technique, remove cap from the drainage tubing and connect directly into the funnel of the urinary catheter.
4. Ensure the connection is secure.
5. Hang the bag at or below the patient's bladder level.

### TO EMPTY THE URINE BAG

1. Perform hand hygiene and don gloves.
2. Place a receptacle directly under the T-Tap of the Urine Bag.
3. Holding the T-Tap in one hand, with fingers on either side of the tap, gently push against the mechanism to slide the tap across the T-connection to open the tap to allow the urine to empty.
4. Once the bag is empty, repeat the above step but in the opposite direction to close the tap.

### PRECAUTIONS

- Ensure the Urine Bag is never raised higher than the level of the bladder.
- Ensure the Urine Bag tubing is free from kinks at all times when connected to the catheter.
- At the completion of emptying the bag, check to ensure the T-Tap is completely closed.
- Ensure aseptic technique is maintained when handling, connecting and disconnecting the Urine Bag from the urinary catheter.

## m|devices

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia  
T +61 (0)2 8718 2800



CONTINENCE  
CARE Urine Bags



**Changshu Taining Medical  
Equipment Co., Ltd**  
Nan Shou Yangyuan Town  
Changshu City, Jiangsu Province  
P.R.China



**MedNet EC-REP GmbH**  
Borkstrasse 10  
48163 Münster  
Germany

