



## UR335010 - UR375030

### Leg Bags

Lever Tap and Silicone Lined Straps



## INSTRUCTION FOR USE

### DESCRIPTION

m|devices Leg Bags are designed to be unobtrusive offering discreet storage under clothing.

The various volume capacity sizes available are 350mL, 500mL and 750mL with three optional tube lengths of 10cm and 30cm with bonded stepped connector, cater for all user requirements.

Lever tap, non-return valves and silicone lined straps with fabric fastener are inclusive in all Leg Bag codes.

**THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL AND OR EDUCATION PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

### CAUTION

- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

### STEPS

1. Remove blue protective cap from stepped connector on the inlet drainage tube and using aseptic technique to connect directly into the funnel end of the Foley Catheter. Ensure the connection is secure.
2. Using the silicone lined straps, apply the Leg Bag to the desired thigh by wrapping the two straps on the bag around the thigh at desired position. Secure the silicone lined straps by pressing together the corresponding side of the fabric fastener on each strap end. Fabric must be firmly pressed together to obtain a secure fixation.
3. To empty the bag (the lever tap will be in the closed position when removed from packaging and moves in both directions by 90 degrees), push the blue lever firmly down towards the outlet tube, this will open the tap and urine will flow out.
4. Once empty, close the tap by pushing the lever firmly up towards the Leg Bag until closed.

### PRECAUTIONS

- Ensure the Leg Bag is never raised higher than the level of the bladder.
- Ensure the Leg Bag tubing is free from kinks at all times when connected to the catheter.
- Ensure aseptic technique is maintained when handling connecting and disconnecting the Leg Bag from the urinary catheter or other drainage bags.
- Ensure connections are always secure between the Foley Catheter and drainage outlet tube.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163

Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices<sup>®</sup>**