

# Reinforced Silicone Laryngeal Mask Airways

## PRODUCT DATA SHEET



### DESCRIPTION

The m|devices laryngeal mask airways are made from 100% medical grade silicone. A smooth one-piece design with guide markings on the handle for easy insertion and positioning. The translucent tubing allows for clear visibility and the soft seal cuff which is permeable to nitrous oxide, reduces pressure rise and minimises potential trauma. The reinforced tube reduces kinking and assists with difficult intubations.

### FEATURES AND BENEFITS

- 100% Medical grade silicone
- Reinforce tube for difficult intubations
- Smooth one-piece design
- Guide markings
- Translucent tubing
- Soft seal cuff
- Peel open pouch
- Colour coded for simple size selection

### PRODUCT DATA

- Material: Silicone
- DEHP: No
- Shelf life: 2 Years
- ARTG Number: 210988
- Single patient use only
- Latex free
- Sterile EO

### PRODUCT PACKAGING LABEL



**INSTRUCTIONS FOR USE**

- Using aseptic technique, take the Laryngeal Mask Airway out of the packaging.
- Deflate the cuff using a syringe and lubricate the anterior surface of the Mask with water-soluble lubricant.
- Hold the Laryngeal Mask Airway at the junction of the cuff and tube.
- With the head extended and neck flexed, carefully insert the Laryngeal Mask Airway to rest on the hard palate.
- Use the Index Finger to push carefully seating pressure on the tube and advance the mask until the resistance is felt at the base of the hypo-pharynx.
- At throat holding the tube, using a syringe inflate the cuff with enough air to make a pressure cuff. Do not over-inflate the cuff.
- In removal of the Laryngeal Mask Airway, use a syringe to deflate the cuff and immediately remove the device.

**CAUTION**

- DO NOT exceed the maximum recommended inflation air volumes.
- DO NOT use the airway if cuff leaks inflate immediately and spontaneously.
- DO NOT re-sterilize or reuse.
- Do not be used by trained personnel.
- DO NOT use if the packaging or product has been damaged or contain liquid.
- DO NOT store at extreme temperatures and humidity. Avoid direct sunlight.

**STERILE EO**

LOT: 0000000  
MFL: yyyy/mm  
MFL: yyyy/mm

Cuff volume: 7mL  
Patient weight: 5-10kg

**m|devices**  
m|devices  
27 Llewellyn Avenue Villawood  
NSW 2163 Australia  
T +61 (0)2 8718 2800  
F +61 (0)2 8718 2801  
info@mdevices.com.au  
mdevices.com.au  
mdevices  
Mulligan Co., Ltd.  
7 Lianjiang Road  
312000 Shanghai China  
CE 0123

RE-ORDER	SIZE	CARTON QTY	RE-ORDER	SIZE	CARTON QTY
AN031000	1	10pcs	AN031004	3	10pcs
AN031001	1.5	10pcs	AN031005	4	10pcs
AN031002	2	10pcs	AN031006	5	10pcs
AN031003	2.5	10pcs			