

Yankauer Suction Handles- Crown tip

PRODUCT DATA SHEET



DESCRIPTION

The m|devices yankauer suction handles consists of suction tip and suction tube. The suction tips are available as a crown tip offering with vent, without vent, or with an on/off switch. Smooth and clean edges for atraumatic suctioning and made from a transparent medical grade PVC.

FEATURES AND BENEFITS

- Made from a transparent PVC
- Crown tip
- Available with vent, without vent, or with on/off switch
- Single and double wrap options
- Peel open pouch
- Sterile EO
- Latex Free

PRODUCT DATA

- Length: 27cm
- Material: PVC (Vent: ABS, PE)
- DEHP: No
- Shelf life: 3 Years
- ARTG Number: 224143
- Outer diameter: 8mm
- Single patient use only
- MRI and CT suitable

PRODUCT PACKAGING LABEL



INSTRUCTIONS FOR USE

1. With the remaining aseptic technique, open the packaging at the connecting end of the Yankauer Handle to expose the connection end only.
2. Connect the Yankauer Handle to the heascan connector on the selected suction tubing.
3. Connect the distal end of the suction tubing to the suction magister.
4. Ensure all connections are secure.
5. Remove packaging from Yankauer Handle and use suction through the Yankauer Handle prior to use.

CAUTION

- Connect with electrocautery electrodes or laser surgery beams must be avoided.
- DO NOT apply continuous suction to mucosal tissue.
- DO NOT use if the packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity. Avoid direct sunlight.

STERILE EO



0000000
yyyy/mm
yyyy/mm



Rev: 02



RE-ORDER	SIZE	CARTON QTY	RE-ORDER	SIZE	CARTON QTY
AN170002	W/out vent single wrap	50pcs	AN170004	O/off switch single wrap	50pcs
AN170002NS	W/out vent single wrap n/sterile	50pcs	AN174002	W/out vent double wrap	50pcs
AN170003	With vent single wrap	50pcs	AN174003	With vent double wrap	50pcs
AN170003NS	With vent single wrap n/sterile	50pcs	AN174004	On/off switch double wrap	50pcs