

Transfusion Pump Set

455cm



PRODUCT DATA SHEET

DESCRIPTION

The Gravity Transfusion Pump Set includes a single vented spike attached to a flexible drip chamber with a 200µm filter. The 455cm kink resistant tubing with male luer lock connector and rotating collar, is made from medical grade PVC. There is a manual hand pump, roller regulator and a swabable needleless access valve.

FEATURES AND BENEFITS

- Vented perforator
- Manual hand pump
- Swabable needleless access valve
- Rigid roller regulator for accurate flow
- Male luer Lock with rotating collar
- Peel open pouch
- Single use only
- Sterile EO
- Latex free

PRODUCT DATA

- Tubing length: 455cm
- Tubing diameter: 3mm
- Drip chamber: 20 drops/mL
- Filter: 200µm
- Priming volume: 72mL
- Material: PVC, PE, ABS, PA, MBS
- DEHP: Yes
- Shelf Life: 5 Years
- ARTG Number: 228375
- The product is MRI and CT suitable

PRODUCT PACKAGING LABEL

Transfusion Set
Manual Pump and Needleless Access Site
Blood / Solution Administration
Length: 455cm

IV030003

STERILE

INSTRUCTIONS FOR USE

1. Remove roller regulator.
2. Remove protective cap from spike and insert into fluid container. In case of rigid container, open vent on spike.
3. Squeeze lower half of the grip chamber until half full.
4. To prime line, open the roller regulator while keeping the Manual Pump and needleless access site, until fluid flows through to the luer connector, expelling all air from the line.
5. Close roller regulator.
6. Remove protective cap from male luer lock connector.
7. Connect and secure by inserting male luer lock connector into access device and engaging rotating collar.
8. Control flow rate with roller regulator.

Using the Needleless Access Site

1. Swab access site with antiseptic before and after use.
2. Connect a luer lock or luer lock connector to the access site by pushing straight into the access site, without using a clockwise rotating motion.

CAUTION

- DO NOT use a needle or blunt cannula with the needleless access site.
- DO NOT use if the packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

LOT: 8888888
EXP: yyyy/mm

m|devices

STERILIZED

CE

RE-ORDER	SIZE	CARTON QTY
IV030003	445cm	50