



Minimum Volume Extension Set

165cm

PRODUCT DATA SHEET



DESCRIPTION

The Minimum Volume Extension Set is a microbore tubing with female to male luer lock ends and rotating collar. There is a variety of lengths available. The Minimum Volume Extension Set is a 165cm tubing with female to male luer lock ends with rotating collar. It includes a click clamp and a swabable Needleless Access Site.

FEATURES AND BENEFITS

- Female to male luer lock with rotating collar
- Click clamp
- Needleless Access Site
- Peel open pouch
- Single use only
- Sterile EO
- Latex free

PRODUCT DATA

- Tubing length: 165cm
- Tubing diameter: 1.5mm
- Priming volume: 3mL
- Material: PVC
- DEHP: No
- Shelf Life: 5 Years
- ARTG Number: 228230
- The product is MRI and CT suitable

PRODUCT PACKAGING LABEL

IV051004

Minimum Volume Extension Set

Female Luer Lock to Male Luer Lock with Rotating Collar
Needleless Access Site
Priming Volume: 3mL
1.5mm ID Tubing (Length: 165cm)

STERILE EO

INSTRUCTIONS FOR USE

1. Remove protective cap from female luer connector.
2. Connect a syringe or IV line with sterile fluid to female luer connector.
3. To prime line, insert needleless access site w/ list allowing fluid to flow through the tubing to the male luer connector to expel all air from the line.
4. Remove protective cap from male luer connector.
5. Connect and secure by inserting male luer connector into access device and engaging rotating collar.

Using the Needleless Access Site

1. Swab access site with antiseptic before and after use.
2. Connect a luer slip or luer lock connector to the access site by pushing straight into the access site, whilst using a clockwise rotating motion.

CAUTION

- DO NOT use a needle or blunt cannula with the needleless access site.
- DO NOT use if the packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

LOT: 8888888

mm/yyyy/mm

yy/yyyy/mm

Rev: 1

m|devices™

Head Office
27 Llewellyn Avenue
Villawood NSW
2163 Australia

T +61(0)2 8718 2800
F +61(0)2 8718 2801
E info@mdevices.com.au
W mdevices.com.au

Head Office
c/o trading name of
Moliplex Medical
Devices Pty Ltd

Hubei Co., Ltd
c/o trading name of
Hubei Moliplex
Medical Devices
Co., Ltd
Room 915-505E, Fox Residences
309 3035 Huaihai East Denmark

CE 1123

RE-ORDER	SIZE	CARTON QTY
IV051004	165cm	100



27 Llewellyn Avenue
Villawood NSW 2163 Australia
T +61(0)2 8718 2800 F +61(0)2 8718 2801

info@mdevices.com.au
mdevices.com.au