


3-Way Silicone Foley Catheter

PRODUCT DATA SHEET

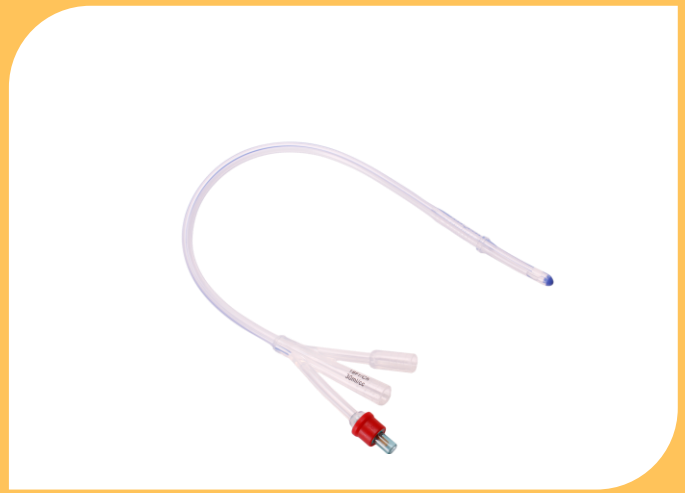

DESCRIPTION

The 3-Way Foley Catheter is made from 100% medical grade translucent and flexible silicone. The symmetrical balloon design supports reliable positioning in the bladder and minimises the potential for ridging.


FEATURES AND BENEFITS

- Urethral and SPC approved
- Symmetric balloon
- Radiopaque
- Long indwelling time 30 days
- Colour coded
- Latex free
- Peel open pouch
- Material: Silicon
- DEHP: No
- Shelf Life: 5 Years
- ARTG Number: 211796
- Sterile EO

PRODUCT DATA

RE-ORDER	DESCRIPTION	TUBING LENGTH	BALLOON CAPACITY	GAUGE	COLOUR
UR013000	3-Way Foley Catheter Standard Tip	40cm	30mL	16Fr	Orange
UR013001	3-Way Foley Catheter Standard Tip	43cm	30mL	18Fr	Red
UR013002	3-Way Foley Catheter Standard Tip	43cm	50mL	20Fr	Yellow
UR013003	3-Way Foley Catheter Standard Tip	43cm	50mL	22Fr	Purple
UR013004	3-Way Foley Catheter Standard Tip	43cm	60mL	24Fr	Blue

PRODUCT PACKAGING LABEL

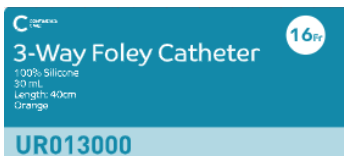

PRECAUTIONS

- Maximum indwelling time is 30 days
- Never clamp or block the catheter or drainage bag tubes whilst 'irrigation' is flowing.
- Ensure the drainage bag is positioned below the level of the bladder and is off the floor.
- The security of all connections should be checked once catheter is connected to drainage bag and 'irrigation fluid' and also monitored during use to prevent disconnections.
- Ensure the catheter is stored in the box and is not coiled or folded when stored.

CAUTION

- DO NOT use with ointments or lubricants with a petroleum base. Use immediately after opening packaging. Discard catheter after use according to hospital protocol.
- DO NOT use if the packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity. Avoid direct sunlight.

STERILE EO



RE-ORDER	DESCRIPTION	CARTON QTY
UR013000	3-Way Foley Catheter Standard Tip	10
UR013001	3-Way Foley Catheter Standard Tip	10
UR013002	3-Way Foley Catheter Standard Tip	10
UR013003	3-Way Foley Catheter Standard Tip	10
UR013004	3-Way Foley Catheter Standard Tip	10