

# Intermittent Closed Catheterisation System

## PRODUCT DATA SHEET



### DESCRIPTION

Our Intermittent Closed Catheterisation System is sterile and pre-lubricated with a hydrophilic gel, so no additional preparation is needed. The closed system works like a traditional Nelaton Catheter and provides a simple catheterisation that reduces the risk of urinary tract infections.


### FEATURES AND BENEFITS

- Sterile closed system
- Pre-lubricated
- Heat polished eyelets
- Smooth round tip
- Colour coded
- Latex free
- Sterile EO
- Bag size: 1500mL
- Side holes: 2
- Length: 42cm
- Material: PVC
- DEHP: No
- BPA: No
- ARTG Number: 211796
- Shelf Life: 3 Years

### PRODUCT DATA

RE-ORDER	DESCRIPTION	LENGTH	BAG CAPACITY	GAUGE
UR050008	Intermittent Closed Catheter	42cm	1500mL	8Fr
UR050010	Intermittent Closed Catheter	42cm	1500mL	10Fr
UR050012	Intermittent Closed Catheter	42cm	1500mL	12Fr
UR050014	Intermittent Closed Catheter	42cm	1500mL	14Fr
UR050016	Intermittent Closed Catheter	42cm	1500mL	16Fr

### PRODUCT PACKAGING LABEL

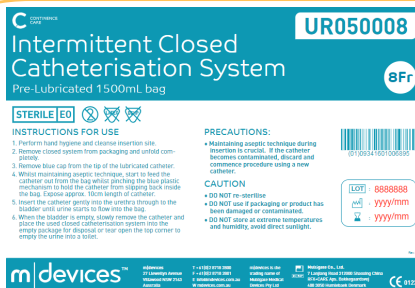

**UR050008**  
Intermittent Closed Catheterisation System  
Pre-Lubricated 1500mL bag  
8Fr

**STERILE EO**

**INSTRUCTIONS FOR USE:**

1. Perform hand hygiene and identify insertion site.
2. Remove closed system from packaging and untitled connector.
3. Remove blue cap from the top of the intermittent catheter.
4. While maintaining aseptic technique, start to feed the catheter out from the bag until reaching the blue plastic mechanism to hold the catheter from slipping back inside the bag. Express airway. Check length of catheter.
5. Insert the catheter gently into the urethra through to the bladder and urine starts to flow into the bag.
6. When the bladder is empty, slowly remove the catheter and close the lock closed catheterisation system from the entire package for disposal or tear open the top corner to empty the urine for a toilet.

**PRECAUTIONS:**

- Maintaining aseptic technique during insertion is critical. If the catheter becomes contaminated, discard and commence procedure using a new catheter.

**CAUTION:**

- DO NOT re-use.
- DO NOT use if packaging or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

LOT: 8888888  
and yyyy/mm  
and yyyy/mm

m|devices

RE-ORDER	GAUGE	CARTON QTY
UR050008	8Fr	25
UR050010	10Fr	25
UR050012	12Fr	25
UR050014	14Fr	25
UR050016	16Fr	25